



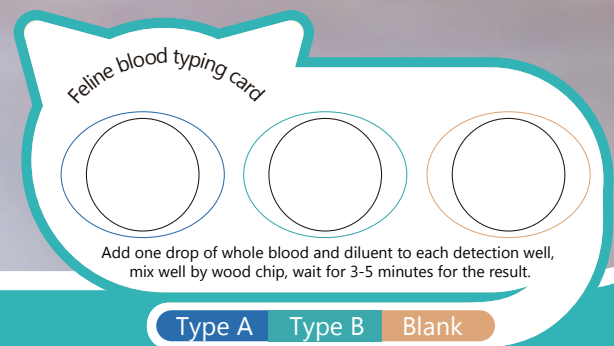

Blood Typing Card (Canine & Feline)

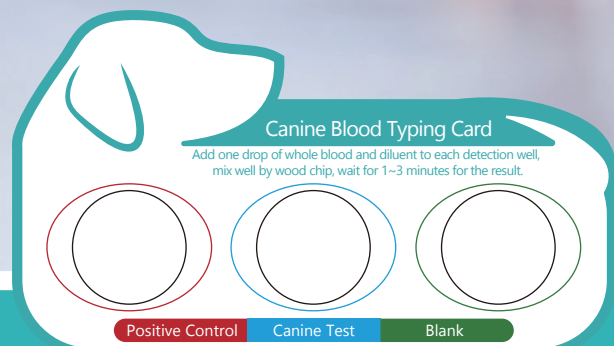

Features



Application Scenarios



Blood transfusion



Breastfeeding



Breeding




Health checking


Test Procedure




Draw 0.5 mL of whole blood into EDTA or alternative anticoagulants tube.

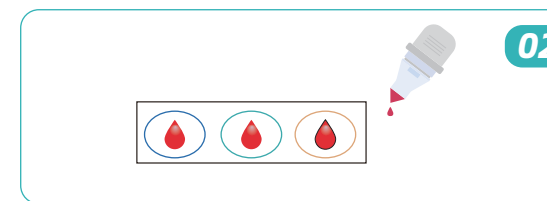

Add one drop of whole blood (around 20 μ L) to 3 test wells.

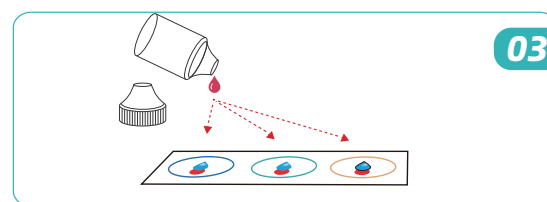

Add one drop of buffer (around 20 μ L) to 3 test wells.

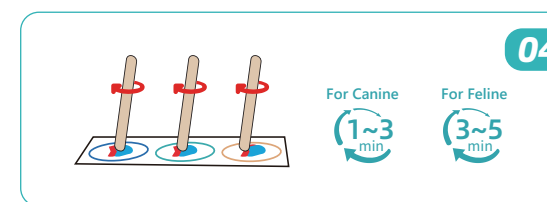

Mix the sample and buffer for 30 seconds, use separate wood chip for each well.

-For Canine Blood Typing Card, check within 1~3min.
-For Feline Blood Typing Card, check with 3~5min.

Interpretation of Results



For Canine

Positive Control	Canine Test	Blank	
			Negative
			Positive

For Feline

Type A	Type B	Blank	
			Type A
			Type B
			Type AB

If other results occur, they are considered invalid and retested is recommended!